

Positioning Checklist for the Mobile Stander

NOTE: Describe all positioning as if you are standing behind the individual

Use this Positioning Checklist as a convenient way to ensure optimal use of the Mobile Stander. Write notes to customize your instructions for each individual.

INDIVIDUAL'S NAME

STAFF NAME

DATE

THIS POSITIONING CHECKLIST IS INTENDED FOR CONSISTENT USE OF ADAPTIVE EQUIPMENT FOLLOWING ASSESSMENT BY A HEALTH PROFESSIONAL. THE CHECKLIST DOES NOT REPLACE AN INITIAL CLINICAL EVALUATION FOR CORRECT SIZING, COMPONENT SELECTION, AND APPROPRIATE THERAPEUTIC POSITIONING FOR THE INDIVIDUAL. SOME COMPONENT SIZING IS DEPENDENT ON THE BASE MODEL SIZE.


Size

- ☐ mini
- ☐ small
- ☐ medium
- ☐ large

Large Wheels

- ☐ on
- ☐ off

☐ Body Support

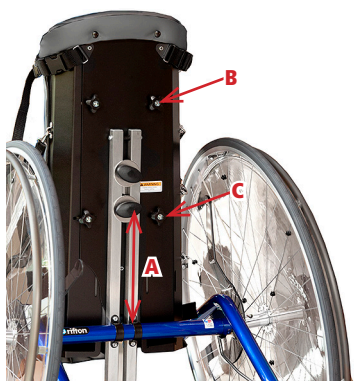

Used as ☐ anterior support ☐ posterior support

Height

Distance from center of lower oval knob to top of colored crossbar (A):

Width

Top: upper lobed knobs (B) adjusted to increase upper body support width

- ☐ yes ☐ no

Bottom: lower lobed knobs (C) adjusted to increase lower body support width

- ☐ yes ☐ no

☐ Fleece Strap



- ☐ Strap shortened and fleece positioned at upper trunk
- ☐ Strap lengthened and fleece positioned behind neck (N/A if in posterior position)
- ☐ Fleece not used

☐ Seat Pad



Size ☐ small ☐ large

Position of attachment straps

Select ring hook position 1–4 (1 is at the top and 4 is at the bottom of the body support.)

Top left strap _____

☐ loose ☐ snug

Top right strap _____

☐ loose ☐ snug

Bottom left strap _____

☐ loose ☐ snug

Bottom right strap _____

☐ loose ☐ snug

☐ Additional Seat Pad



Size ☐ small ☐ large

Position of attachment straps

Select ring hook position 1–4 (1 is at the top and 4 is at the bottom of the body support.)

Top left strap _____

☐ loose ☐ snug

Top right strap _____

☐ loose ☐ snug

Bottom left strap _____

☐ loose ☐ snug

Bottom right strap _____

☐ loose ☐ snug

☐ **Body Support Center strap**



- ☐ Buckled to the seat pad (threaded between user's legs)
- ☐ Not used

☐ **Leg Straps** (on body support) ☐ Snug ☐ Loose ☐ Not used

☐ **Ankle Straps** (on footplate) ☐ Snug ☐ Loose ☐ Not used

☐ **Sandals**



☐ **Left**

Distance between sandal toe and side of footplate: _____

Distance between sandal heel and side of footplate: _____

Distance between sandal heel and back edge of footplate: _____

☐ **Wedge**

- ☐ Thick edge at heel
- ☐ Thick edge at toe
- ☐ Stacked for level lift

☐ **Right**

Distance between sandal toe and side of footplate: _____

Distance between sandal heel and side of footplate: _____

Distance between sandal heel and back edge of footplate: _____

☐ **Wedge**

- ☐ Thick edge at heel
- ☐ Thick edge at toe
- ☐ Stacked for level lift

☐ **Arm Prompts** (require mounting bracket)

Size ☐ small ☐ large



☐ **Left**

Clamp position (L-bar holder)

☐ toward body support ☐ away from body support

Distance of clamp from end of mounting bracket: _____

L-bar post position

☐ points away from body support ☐ points right ☐ points left

L-bar height Specify number of notches visible above clamp: _____

Orientation of arm prompt ☐ horizontal ☐ angled up ☐ angled down

Rotation position (specify as o'clock) _____

Hook and loop strap

☐ across forearm ☐ behind elbow ☐ no strap

☐ **Right**

Clamp position (L-bar holder)

☐ toward body support ☐ away from body support

Distance of clamp from end of mounting bracket: _____

L-bar post position

☐ points away from body support ☐ points right ☐ points left

L-bar height Specify number of notches visible above clamp: _____

Orientation of arm prompt ☐ horizontal ☐ angled up ☐ angled down

Rotation position (specify as o'clock) _____

Hook and loop strap

☐ across forearm ☐ behind elbow ☐ no strap

..... **OR**

☐ **Hand Loops** (require mounting bracket)



☐ **Left**

Orientation of hand loop ☐ vertical ☐ tilted forward ☐ tilted backward

Clamp position (hand loop holder)

☐ toward body support ☐ away from body support

Distance of clamp from end of mounting bracket: _____

Hand loop height Specify number of notches visible above clamp: _____

☐ **Right**

Orientation of hand loop ☐ vertical ☐ tilted forward ☐ tilted backward

Clamp position (hand loop holder)

☐ toward body support ☐ away from body support

Distance of clamp from end of mounting bracket: _____

Hand loop height Specify number of notches visible above clamp: _____

☐ **Communication Tray** (requires mounting bracket)



Clamp position

☐ to user's left

☐ facing toward body support

☐ facing away from body support

☐ to user's right

☐ facing toward body support

☐ facing away from body support

☐ Centered (can only face away from body support)

Post height

Specify number of notches visible above clamp: _____